

Roche CoaguChek Pro II - INR Meter

This Ward Handbook has been designed to ensure that the CoaguChek Meter is being used within the governance guidelines of the LTHT Point of Care Policy.

Contact Details

The Point of Care Testing (POCT) Team can be contacted between the hours of Mon-Sat 08:30-17:00 on:

Ext: 22338 or 64791

Mobile: 07775996028

Email: leedsth-tr.pointofcare@nhs.net

Consumables

CoaguChek® PT test strips 1 x 48 06688721019

CoaguChek® PT Controls 2 x 4 06679684190

Safe-T-Pro Plus single use device 1 x 200 03603539150

Please order consumables through your supplies department.

Training

Training sessions can be arranged directly with the clinical area Key trainer who can be identified by contacting POCT. Alternatively training can be organised directly with POCT.

The CoaguChek Operator Guide [[BSF2POC123](#)] is available on the POCT website.

Documentation

Please complete the sheets within this Ward Handbook, compliance will be audited on a 12 monthly basis.

Reagent Register - Product information for new QC and Strip Lot deliveries

Sample Record - To record sample and QC results

Quality Control Record - Two levels of QC performed weekly

Quality Control Record - Level 1

Liquid QC samples must be run on a weekly basis, and all results recorded below. Failure to do so will result in the meter becoming locked and unavailable for use until the QC sample has been run.

Analyser serial number _____

Date	Time	Test Strip		QC		Result	Target Range	Pass/Fail (If fail - action)	Operator
		Lot NO.	Expiry Date	Lot NO.	Expiry Date				

Copies without an Authorised stamp or document manager signature are no longer valid after 7 days of this date 17/01/2024

Quality Control Record - Level 2

Liquid QC samples must be run on a weekly basis, and all results recorded below. Failure to do so will result in the meter becoming locked and unavailable for use until the QC sample has been run.

Analyser serial number _____

Date	Time	Test Strip		QC		Result	Target Range	Pass/Fail (If fail - action)	Operator
		Lot NO.	Expiry Date	Lot NO.	Expiry Date				

Copies without an Authorised stamp or document manager signature are no longer valid after 7 days of this date 17/01/2024

**QPulse Number: POCT-SOP-11
v 1.2**

Title: CoaguChek Ward Handbook

Page 6 of 6

Copies without an Authorised stamp or document manager signature are no longer valid after 7 days of this date 17/01/2024